

ALARMED DEFIBRILLATOR CABINET

This alarmed cabinet deters would be thieves as well as announcing an emergency as soon as the door is opened. The battery operated alarm will sound once the door is opened.

The AED is visible through the perspex cabinet front.

Model number: Y4611990

Size:

Depth:

External 200mm

Internal 176mm

Width:

External 415mm

Internal 405mm

Height:

External 380mm

Internal 370mm

Colour: White


Warranty: 12 months



INSTALLATION GUIDE AND OPERATING INSTRUCTIONS

1. Hold the wall cabinet at a suitable height
2. Mark on the wall the positions of the 4 fixing holes, visible from the inside of the cabinet
3. Secure the cabinet using appropriate fixing methods through these holes
4. Place operating AED in alarm cabinet
5. Switch on alarm (see *figure 1*) and check operation by opening the door and closing it again. The cabinet installation is now complete

Fig 1

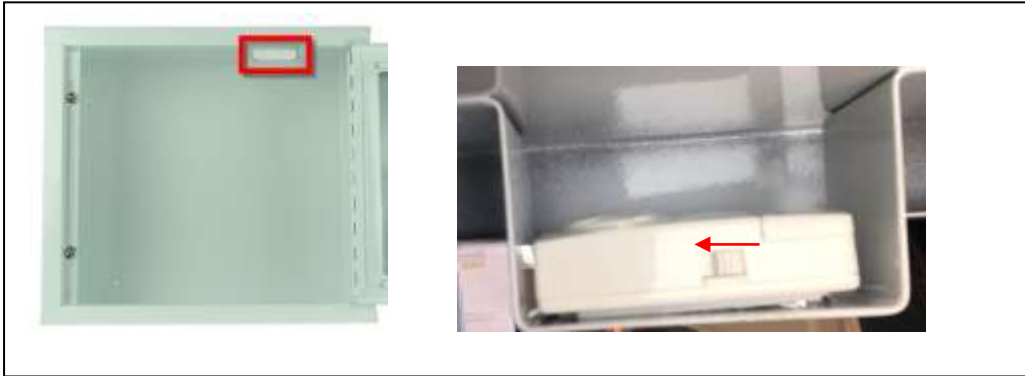

TROUBLE SHOOTING

Battery replacement should only be necessary after several years. A simple monthly check is to briefly open and close the cabinet door to ensure that the alarm is functioning properly. To replace the battery, see below instructions

REPLACING THE BATTERY

Instructions for replacing the battery:


1) Locate battery compartment



2) Lift out of battery compartment



3) Slide off battery cover

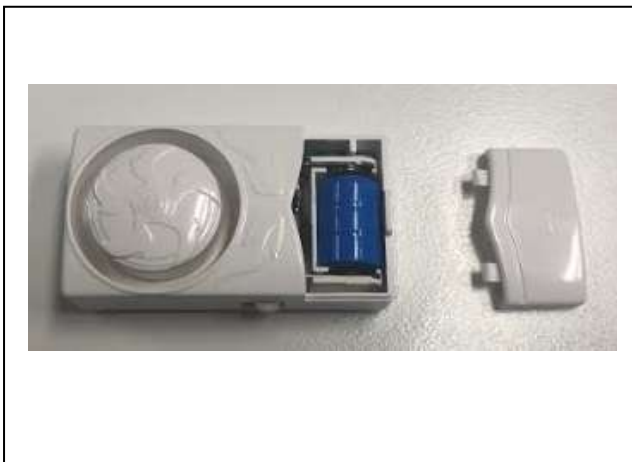

4) Replace battery

