

Connecting your LIFEPAK® CR2 cellular device

Refer to: *Getting Started Guide* and *Wireless Setup Guide* that comes packaged with device



- 1 Take the device to its intended location where the device will be installed.
- 2 Insert battery and wait for self-test to complete. May take up to 2 minutes. **Wait until 3 audible tones sound before proceeding to next step.**

Note: The battery included with the device may have less than the standard 4-year service life. However, the battery and device will function as intended until the battery's expiry.



- 3 Verify the **readiness indicator is flashing green** to indicate readiness (flashes every 6 seconds).

- 4 **Open the lid and wait** until the voice prompts start.

- 5 Immediately **press and hold the LANGUAGE and CHILD MODE buttons** simultaneously to complete a self-test until you hear **"Device ready."** After the voice prompts start, you must press the 2 buttons within 10 seconds.

- 6 **Immediately close the lid** and the device will attempt a wireless connection. The device must be in its permanent location during this test.

- 7 Wi-Fi will try to be connected first—if you do not connect it to a Wi-Fi network, this will fail. You will hear some prompts, pauses, and tones during this process.

- 8 Once the Wi-Fi connection fails, **cellular connection will begin.** This occurs whether or not the Wi-Fi connection was successful.

Once device is connected to a cellular connection, you will hear these prompts:

"Mobile connection initiated"
"Connection in progress"
"Connection established"
"Communication complete"
"Powering off"

- 9 Your device will be connected and ready for use.



ALWAYS FOLLOW THE DIRECTIONS FOR USE

The information presented is intended to demonstrate Stryker's product offerings. Refer to operating instructions for complete directions for use indications, contraindications, warnings, cautions, and potential adverse events, before using any of Stryker's products. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your representative if you have questions about the availability of Stryker's products in your area. Specifications subject to change without notice.

Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: LIFEPAK and Stryker. All other trademarks are trademarks of their respective owners or holders. The absence of a product, feature, or service name, or logo from this list does not constitute a waiver of Stryker's trademark or other intellectual property rights concerning that name or logo.

CA5208_SP

© 2025 Stryker

Stryker Australia Pty Ltd
 8 Herbert St,
 St Leonards NSW 2065
 T: +61 2 9467 1000
 F: +61 2 9467 1010
 stryker.com

Stryker New Zealand Limited
 511 Mount Wellington Hwy,
 Mt Wellington Auckland 1060
 T: +64 9 573 1890
 F: +64 9 573 1891